|   |                     | B2020-40STFX | B2020-40STMX | B2040TI   |
|---|---------------------|--------------|--------------|-----------|
| 1 | Flat point screw M8 | V03I-0816    | V03Z-0816    | V03I-0812 |

### Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

| Screw                   | Torques (Nm) | Screw     | Torques (Nm) |
|-------------------------|--------------|-----------|--------------|
| V03I-0816 and V03I-0812 | 13           | V03Z-0816 | 15           |

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

### Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- Use a damp cloth to clean the product

### Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

### EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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## B2020-40STFX/STMX/TI

### Designation

Tubular adapter with light alloy tube  $\varnothing$  30 mm

- |                               |                |                 |
|-------------------------------|----------------|-----------------|
| ⌚ B2020STFX : Light alloy     | Weight : 193 g | Length : 200 mm |
| ⌚ B2040STFX : Light alloy     | Weight : 333 g | Length : 400 mm |
| ⌚ B2020STMX : Stainless steel | Weight : 236 g | Length : 200 mm |
| ⌚ B2040STMX : Stainless steel | Weight : 358 g | Length : 400 mm |
| ⌚ B2040TI : Titanium          | Weight : 308 g | Length : 400 mm |

### Purpose

Structural element used to make endoskeletal prostheses of the lower limbs

- ⌚ B2020STFX et B2040STFX : Weight limit : 100 kg
- ⌚ B2020STMX et B2040STMX : Tested in accordance with ISO 10328 P6 136 kg
- ⌚ B2040TI : Weight limit : 136 kg

### Adjustments

Tilt amplitude : +/- 9° compared to the reference axis.

Set the desired tilt using the 4 screws, in accordance with the limits set.

Lock the assembly using (Ref. : Z31-243) locking compound on the screws.

The tube can be cut again in length according to your needs.

Do not grip the tube in a vise !

Use tube cutter (Ref. : ZOC103) by applying it in comparison with the tube at right angles. Deburred carefully inside and outside of the tube with the reamer (Ref. : ZOC104).

Insert the tube to the stop of the adapter.