Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screws	Torques (Nm)
V02I-0620 (3)	10
V03TI-0616 (4)	5
V02I-0825 (5)	15

- ⌚ Assemble the screws using normal (Ref. : Z31-243) locking compound
- ⌚ Mount the tube on the stop of the bore bottom
- ⌚ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⌚ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⌚ All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- ⌚ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⌚ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- ⌚ Use a damp cloth to clean the product

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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Flexion/extension control

Adjustment of swing phase :

- Regulator (2) : screw (clockwise) raises the extension's return resistance

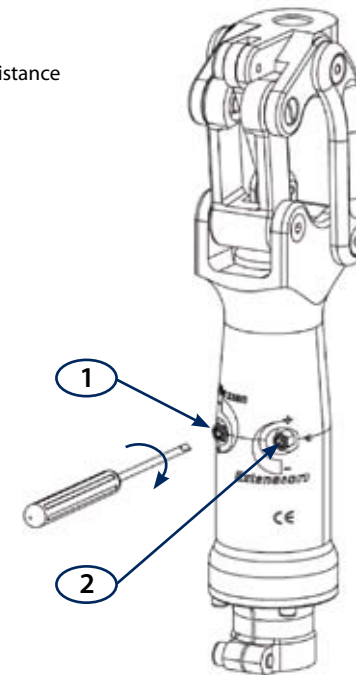
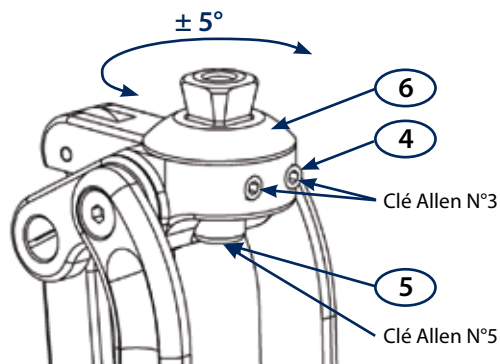
Adjustment of stance phase :

- Regulator (1) : screw (clockwise) raises the flexion's resistance


When screwing regulator, never tighten the screw completely.

Adjustment of proximal attachment

Angulation maximum : $\pm 5^\circ$

Spares parts

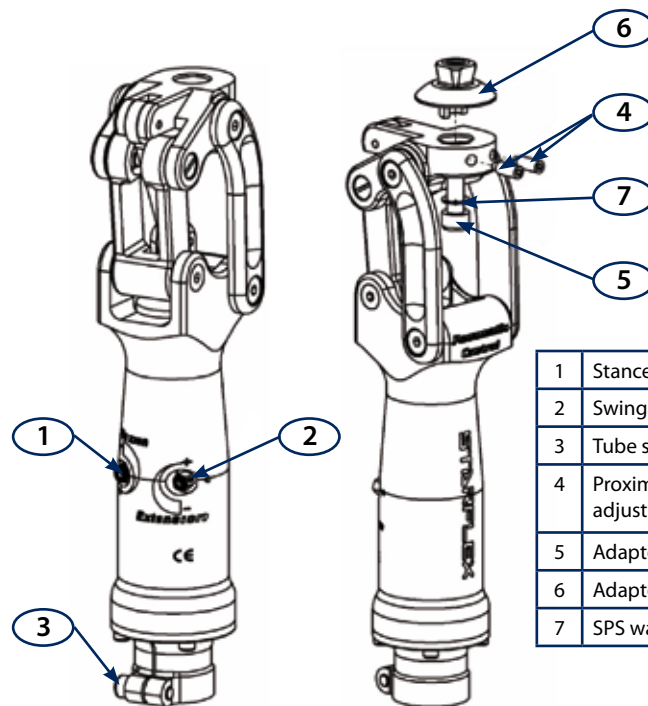1	Stance phase regulator	
2	Swing phase regulator	
3	Tube screw	V02I-0620
4	Proximal attachment adjustment screw	V03TI-0616
5	Adaptor screw	V02I-0825
6	Adaptor	
7	SPS washer	V21-081313

Compatibility

- Proximal attachment : connector of the STARFLEX range
- Distal attachment : Ø 30 mm tube

Connectors in the STARFLEX range (upper section)

CA1
Male pyramid


CA3
Connector
for temporary fitting

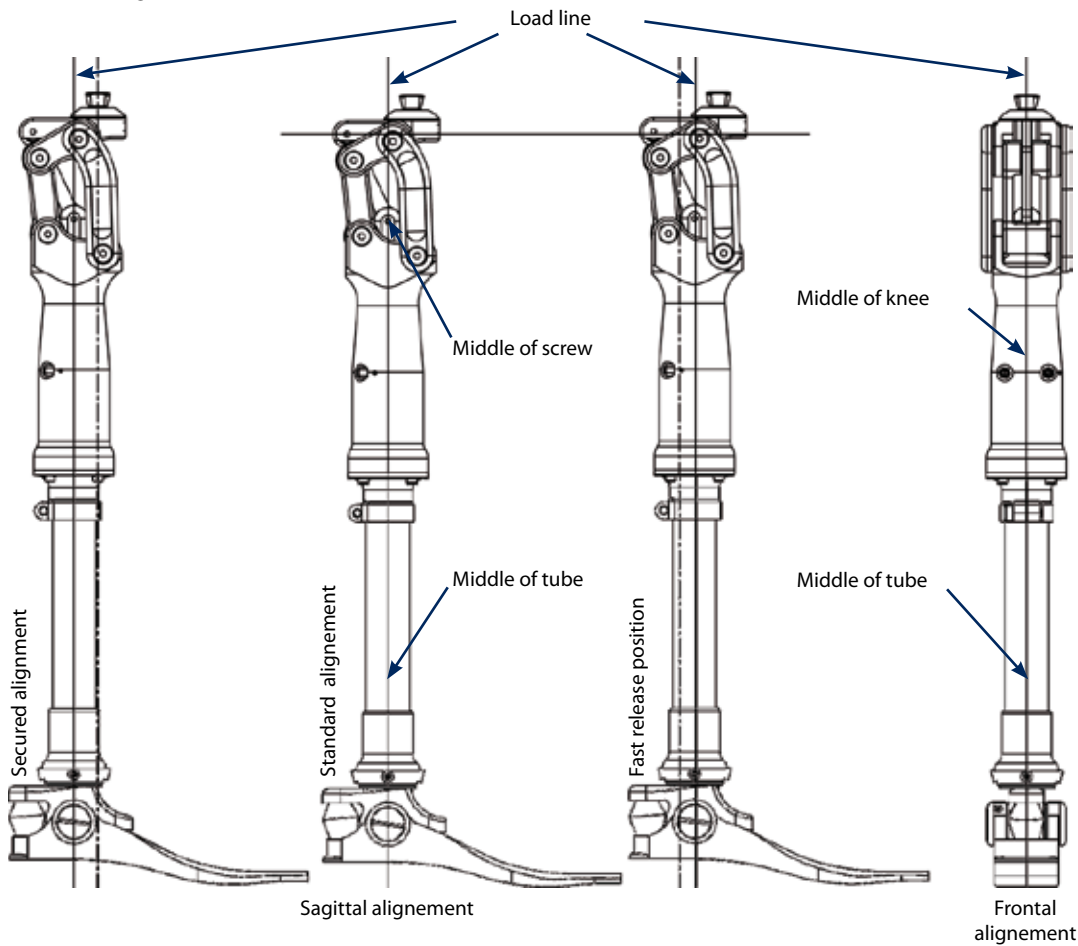CA7
Rotary adaptor with anchor
for cementing, large branches

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