Set the desired tilt using the 4 screws, in accordance with the limits set.
 Lock the assembly using LOCTITE 243 (ref Z31-243) locking compound on the screws V03I-0812 only.
 Screw tightening torque (use key ZOC102) :

Screws	Torque (Nm)
V03I-0812	5
VO3PI-0516	3
V02Z-0516	9

Spares parts

1	Flat point screw M8 L12	V03I-0812
2	Screw cone point M5 L16	V03PI-0516
3	Flat point screw M5 L16	V02Z-0516

Restrictions on use / Precautions for use

- Assemble the screws using normal LOCTITE 243 locking compound
- The adaptor must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use (patient's activities, environment, etc...)
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Store the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- Use a damp cloth to clean the product

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B.: In the event of a complaint, please enclose a return form correctly filled in; incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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TITANE

Designation

Sliding double receiver in titanium.

- ↪ Weight : 149 g

Purpose

Structural element used to make endoskeletal prostheses of the lower limbs for patients having average levels of activity and whose weight does not exceed 136 kg.

Compatibility

The tubular adaptors are compatible with all the tubes and adaptors in the Titanium range.

Fitting

Adjustment

- ↪ Tilt amplitude: +/- 9° compared to the reference axis
- ↪ Offset amplitude: +/- 10 mm

Offset adjustment

- ↪ The patient should not be put in charge of the prosthesis
- ↪ Use a 2,5 mm hex key to adjust the translation

Attention !

Never remove the safety-lock set screw V03PI-0516.