## Fitting

- ⤿ Adjust the amount of the upright
- ⤿ On the upright of the orthosis, drill and countersink the holes  $\varnothing 4$
- ⤿ Fasten the upright with spindle with screws countersunk head

## Adjustment

Move the upright with spindle and keep at least 4 screws

## Restriction on use / Precautions for use

- ⤿ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⤿ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⤿ All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- ⤿ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⤿ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

## EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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