Composition

A pair of metal fittings with reserve on the lower and upper uprights in semi-hardened steel.

Spares parts

	M40	M41	M42
Articulation screw (head diameter)	V0111 (T14)	V0124 (T12)	V0124 (T12)
Articulation screw (head diameter)	V0119 (T16)	V0117 (T12)	V0152 (T14)
Hoffa lock external right / internal left	M4005ED	M4105ED	M4205ED
Hoffa lock external left / internal right	M4006EG	M4106EG	M4206EG

Restriction on use / Precautions for use

- ⌚ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⌚ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⌚ All assembly and adjustment work must be carried out by qualified personnel
- ⌚ Heat the part with a blowtorch before modifying its shape (any cold shaping will nullify the guarantee)

Cleaning / Storage

- ⌚ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⌚ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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