1	Articulation screw	V0159-1256
2	Bronze ring	M9108-1
3	FHC screw M4 L10	V05I-0410
4	FHC screw M4 L8	V05I-0408
5	FHC screw M3 L8	V05I-0308
6	Convex cruciform screw	V52I-0306

Compatibility

- All the orthotics components of JANTON range

Fitting

On the adjustable splint, the assembly between the leg part and the dial must be realized with the screw (6).

Adjustment

On the adjustable splint, the two M3 threaded holes made in the amount of leg allows adjustment every 6°. Amplitude : 6° recurvatum at 36° flexion (8 positions).

Restriction on use / Precautions for use

- Don't heat the knee joint
- Assemble the screws using normal Z31-243 (LOCTITE 243) locking compound
- Take care not to deform the mechanism during the bending
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

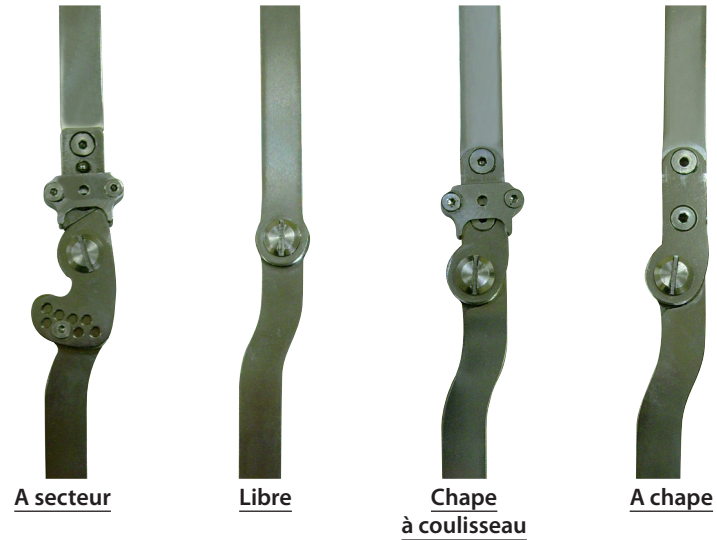
Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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